





Recliner

Permanent Back Attachment



Step 1. Remove the back from the recliner. To do so, lift the “skirt” on the bottom of the back. It is held in place by hook and loop fastener and will easily pull away from the cross brace.

Step 2. Lift the tabs of the metal receivers (attached to the back) away from the steel bayonet on the base mechanism as shown in Figure 2. (It may take one person on each side.)

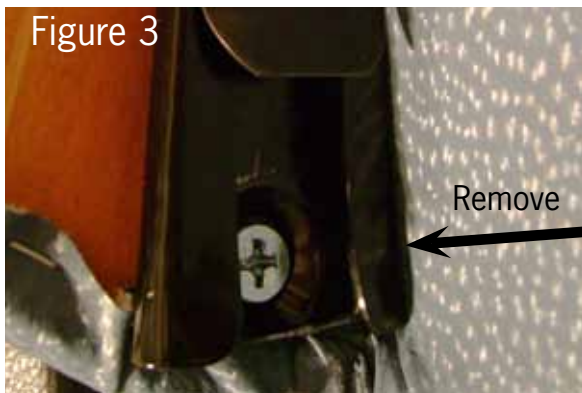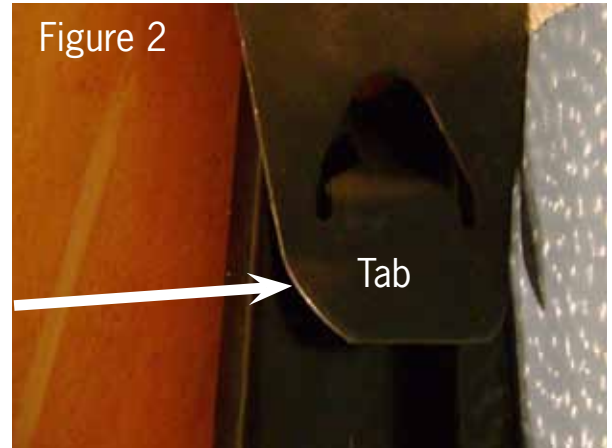

Step 3. Using a phillips screwdriver, remove the bottom screw in the receiver as shown in Figure 3.

Step 4. Reinstall the back onto the bayonet on the base mechanism as shown in Figure 4.



Step 5. Install the new longer screws through the bayonet on the base mechanism and through the open hole of the receiver in the back as shown in Figure 5.

Goes Great with Work.®

