

Installation Instructions Pillow for Recliner



There are two (2) straps factory-attached to the pillow. These straps will loop over the top of the recliner as shown in the photo at left.

These straps have “hook and loop” fastener strips (commonly referred to as Velcro®) that are also factory installed. Make certain that the straps are pulled toward the closest edge of the pillow to properly hang on the recliner.



The straps will loop over the push bar on the back of the recliner. Place the end of the strap against the middle of the strap to engage the hook and loop fastener. There is a sufficient length of the fastener material to allow the adjustment of the pillow up and down for optimum comfort.